Asahi**KASEI**

**IMMUSORBA™**

**TR-350(L)**

**İmmünoadsorpsiyon Kolonu**

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
İşlenen plazma hacmi: 2 L

N.Shibuya, et al. Current Practice in Therapeutic Plasmapheresis 166-172, 1985

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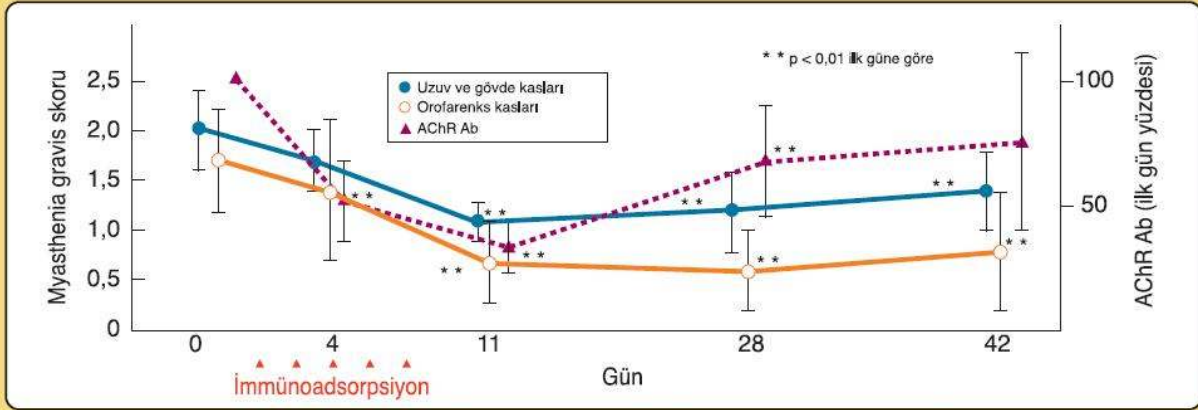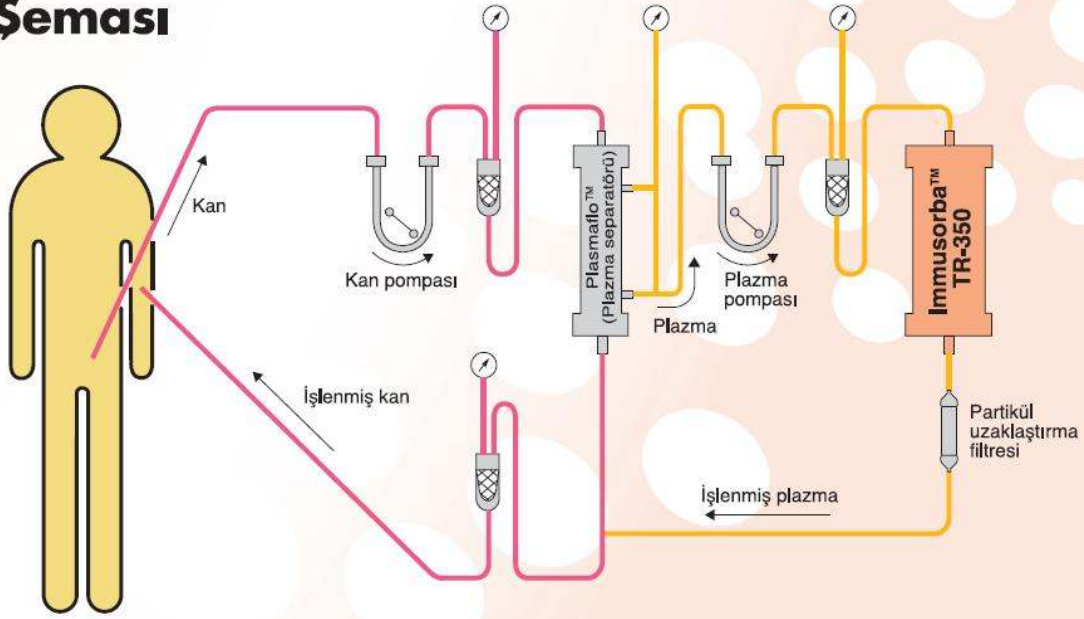
# Referanslar

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- 1) Hirata N, Kuriyama T, Yamawaki N. Immusorba TR and PH. Ther Apher Dial. 7:85-90, 2003
- 2) Yoshida M, Tamura Y, Yamada Y, et al. Immusorba TR and Immusorba PH: basics of design and features of functions. Ther Apher. 4:127-134, 2000

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- 1) Sulowicz W, Stompor T. LDL-apheresis and immunoadsorption: novel methods in the treatment of renal diseases refractory to conventional therapy. Nephrol Dial Transplant. 18 Suppl 5:v59-62, 2003
- 2) Yang KS, Kenpe K, Yamaji K, et al. Plasma adsorption in critical care. Ther Apher. 6:184-188, 2002
- 3) Tagawa Y, Yuki N, Hirata K. Ability to remove immunoglobulins and anti-ganglioside antibodies by plasma exchange, double-filtration plasmapheresis and immunoadsorption. J Neurol Sci. 157:90-95, 1998
- 4) Yamazaki Z. Extracorporeal immunoadsorption. Therapeutic Plasmapheresis VI 113-121, 1986

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- 1) Yamazaki Z, Idezuki Y, Inoue N, et al. Extracorporeal immunoadsorption with IM-PH or IM-TR column, Biomater Artif Cells Artif Organs. 17:117-124, 1989
- 2) Tsuruta Y, Maeda K, Shinzato T, et al. Clinical Application of Improved I-02 Immunoadsorbent Column. Plasmapheresis ISAO Press Cleveland. 149-154, 1983
- 3) Yamazaki Z, Fujimori Y, Takahama T, et al. Efficiency and biocompatibility of a new immunosorbent. Trans Am Soc Artif Intern Organs. 28:318-323, 1982

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- 1) Splendiani G, Cipriani S, Passalacqua S, et al. Plasmapheresis on triptophan columns can improve the clinical outcome of patients affected with myasthenia gravis. Artif Cells Blood Substit Immobil Biotechnol. 31:69-79, 2003
- 2) Haupt WF, Rosenow F, van der Ven C, et al. Immunoadsorption in Guillain-Barre syndrome and myasthenia gravis. Ther Apher. 4:195-197, 2000
- 3) Batocchi AP, Evoli A, Di Schino C, et al. Therapeutic apheresis in myasthenia gravis. Ther Apher. 4:275-279, 2000
- 4) Chiu HC, Chen WH, Yeh JH. The six year experience of plasmapheresis in patients with myasthenia gravis. Ther Apher. 4:291-295, 2000
- 5) Yeh JH, Chiu HC. Comparison between double-filtration plasmapheresis and immunoadsorption plasmapheresis in the treatment of patients with myasthenia gravis. J Neurol. 247:510-513, 2000
- 6) Yeh JH, Chiu HC. Plasmapheresis in myasthenia gravis. A comparative study of daily versus alternately daily schedule. Acta Neurol Scand. 99:147-151, 1999
- 7) Yeh JH, Chiu HC. Optimal volume of processed plasma and total number of selective plasmapheresis sessions in the treatment of patients with severe generalized myasthenia gravis. J Clin Apher. 14:177-180, 1999
- 8) Grob D, Simpson D, Mitsumoto H, et al. Treatment of myasthenia gravis by immunoadsorption of plasma. Neurology. 45:338-344, 1995
- 9) Shibuya N, Sato T, Osame M, et al. Immunoadsorption therapy for myasthenia gravis. J Neurol Neurosurg Psychiatry. 57:578-581, 1994
- 10) Sawada K, Malchesky PS, Koo AP, et al. Myasthenia gravis therapy: immunoadsorbent may eliminate need for plasma products. Cleve Clin J Med. 60:60-64, 1993
- 11) Avanzi G, Marconi G. Semiselective Immunoadsorption of anti-AchR abs on Tryptophan column in Myasthenia Gravis. Clinical Experience in 32 patients. Transfus Sci. 14:17-21, 1993
- 12) Splendiani G, Passalacqua S, Barbera G, et al. Semi-selective immunoadsorption treatment in myasthenia gravis. Biomater Artif Cells Immobilization Biotechnol. 20:1145-1151, 1992
- 13) Splendiani G, Passalacqua S, Barbera G, et al. Myasthenia gravis (MG) treatment with immunoadsorbent columns. Biomater Artif Cells Immobilization Biotechnol. 19:255-265, 1991
- 14) Sugahara S, Yoshikawa Y, Saitoh K, et al. Clinical Evaluation of Immunoadsorbent Therapy and Double Filtration Plasmapheresis in Refractory Myasthenia Gravis. Therapeutic Plasmapheresis (X) 111-115, 1991
- 15) Shibuya N, Jabazawa H, Osame M, et al. Immunoadsorption Therapy for Myasthenia Gravis: Multicenter Trial in a Single Blind Design. Therapeutic Plasmapheresis (VIII):231-233, 1989

- 16) Passalacqua S, Splendiani G, Sturniolo A, et al. Plasma perfusion in myasthenia gravis. ASAIO Trans. 34:602-605, 1988
- 17) Sato T, Ishigaki Y, Komiya T, et al. Therapeutic immunoadsorption of acetylcholine receptor antibodies in myasthenia gravis. Ann N Y Acad Sci. 540:554-6, 1988
- 18) Heinger K, Toyka KV, Borberf H. Selective Removal of Antibodies : Theoretical and Practical Aspects. Therapeutic Plasmapheresis(VI) 136-142, 1986
- 19) Shibuya N, Nagasato K, Kinoshita N, et al. Immunoadsorbent perfusion therapy in patients with myasthenia gravis. Current Practice in Therapeutic Plasmapheresis 166-172, 1985
- 20) Shibuya N, Nagasato K. New treatment using immunoadsorption column. [Article in Japanese] Chiryō. 66:49-55, 1984
- 21) Sato T, Anno M, Arai K, et al. In vitro removal of anti-acetylcholine receptor antibodies with a new immunoadsorbent in sera from myasthenia gravis patients. Progress in Artificial Organs 719-722, 1983

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- 1) Okamiya S, Ogino M, Ogino Y, et al. Tryptophan-immobilized column-based immunoadsorption as the choice method for plasmapheresis in Guillain-Barre syndrome. Ther Apher Dial. 8:248-253, 2004
- 2) Diener HC, Haupt WF, Kloss TM, et al. A preliminary, randomized, multicenter study comparing intravenous immunoglobulin, plasma exchange, and immune adsorption in Guillain-Barre syndrome. Eur Neurol. 46:107-109, 2001
- 3) Haupt WF, Birkmann C, van der Ven C, et al. Apheresis and selective adsorption plus immunoglobulin treatment in Guillain-Barre syndrome. Ther Apher. 4:198-200, 2000
- 4) Haupt WF, Rosenow F, van der Ven C, et al. Sequential treatment of Guillain-Barre syndrome with extracorporeal elimination and intravenous immunoglobulin. Ther Apher. 1:55-57, 1997
- 5) Haupt WF, Rosenow F, van der Ven C, et al. Sequential treatment of Guillain-Barre syndrome with extracorporeal elimination and intravenous immunoglobulin. J Neurol Sci. 137:145-149, 1996
- 6) Yamazaki M, Koh CS, Hanyu N, et al. Comparative study of the efficacy of plasma exchange, immunoadsorption plasmapheresis and corticosteroid administration in the treatment of Guillain-Barre syndrome. Article in Japanese with English Abstract] Aterugi. 44:498-502, 1995
- 7) Haupt WF, Borberf H, Rosenow F. Pilot study of a new treatment concept for acute Guillain-Barre syndrome. Transfus Sci. 16:139-144, 1995
- 8) Jimenez C, Rosenow F, Grieb P, et al. Adsorption therapy with tryptophan-conjugated polyvinyl alcohol gels in 10 patients with acute Guillain-Barre syndrome. Transfus Sci. 14:9-11, 1993
- 9) Rosenow F, Haupt WF, Grieb P, et al. Plasma exchange and selective adsorption in Guillain-Barre syndrome—a comparison of therapies by clinical course and side effects. Transfus Sci. 14:13-15, 1993
- 10) Iwahashi M, Maeda T, Ohnishi K, et al. Immunoadsorption (TR-350) in Guillain-Barre Syndrome. Therapeutic Apheresis (XII) 551-555, 1993
- 11) Iwahashi M, Maeda T, Ohnishi K, et al. Immunoadsorption (TR-350) in Guillain-Barre Syndrome. Therapeutic Apheresis (X) 88-93, 1991

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- 1) Beige J, Perez-Canto A, Moosmayer I, et al. Immunoadsorption with tryptophan adsorbents for successful treatment of late steroid-refractory recurrent focal glomerulosclerosis. Am J Transplant. 3:1459, 2003
- 2) Widder RA, Brunner R, Engels B, et al. Changes of hemorheological and biochemical parameters after plasma perfusion using a tryptophan-polyvinyl alcohol adsorbent leading to clinical improvement in patients suffering from maculopathy. Blood Purif. 16:15-21, 1998
- 3) Griffin S, Chapman P, Elliott J, et al. Recombinant Human Autoantigens as an Extracorporeal Immunoadsorbent in Therapeutic Apheresis. Jpn J Apheresis 16:17-22, 1997
- 4) Yuki N. Tryptophan-immobilized column adsorbs immunoglobulin G anti-GQ1b antibody from Fisher's syndrome: A new approach to treatment. Neurology. 46:1644-1651, 1996
- 5) Yamada H, Takamori K, Ogawa H. In vitro study on the removal of pemphigus antibody with immunoadsorption. J Clin Apher. 11:14-15, 1996
- 6) Koh CS, Shimada K, Inoue A. Immunoadsorption Plasmapheresis in Polymyositis/ Dermatomyositis : Beneficial Effects in Four Cases. Jpn J Apheresis 13:131-132, 1994

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## ASAHI KASEI KURARAY MEDICAL CO., LTD.
## ASAHI KASEI MEDICAL EUROPE GmbH